

## INSTRUCTION FOR USE

### Product Name: Flat Sterilization Pouch

#### Product Description:

Designed as a packaging solution for sterilization applications, MEDSTER SafeCheck, Flat Sterilization Pouches ensure the safe protection of the medical device/material from microorganisms from post-sterilization to use.







With its wide range sizes, it facilitates the selection of the appropriate packaging for your medical supplies. LOT number, Manufacturing Date and Expiry Date can be found on the product label. Our product is made of PET/PP copolymer film and medical kraft paper. Ethyleneoxide (EO), Steam and Formaldehyde (FO) indicator printing may be present on it. Our products meet the performance specifications specified in ISO 11607, ISO 11140-1 and EN 868-5 standards

The shelf life of our product is 5 years.

#### Instruction:

- Place the medical device/material to be sterilized inside the MEDSTER SafeCheck Flat Sterilization Pouch
- Do not fill pouches more than 75% of their capacity.
- For proper airflow, always leave a gap of at least 1.5 cm between the medical device/material to be sterilized and each weld edge of the pouch.
- Seal the open side of the pouch with a calibrated and approved sealing machine.
- When loading into the sterilizer, place the pouches in the basket with film facing film and paper to paper.
- Do not use wet pouches. Repeat the sterilization process after replacing the wet pouches.
- For a clean peel, always open the pouch slowly and with the opening direction indicated on the product.
- Do not use if the bag is torn, has not made the specified color change or is damaged.

#### Indicator Features:

Sterilization Method	Before Sterilization	After Sterilization
Ethylene Oxide	Blue	Yellow
		
Steam	Pink	Brown/Black
		
Formaldehyde	Red	Green
		





















## INSTRUCTION FOR USE

### Storage, Storing and Disposal Information:

- Do not expose to direct sunlight and keep it dry.
- Between 0°C and 40°C and Max. Store at 70% RH humidity.
- Store in its original packaging.
- Dispose in accordance with the place of use and local regulations.

### Symbols and their meanings on the product label and user manual:

Symbols	Meaning	Symbols	Meaning	Symbols	Meaning
	Manufacturer Information		Production Date (Month/Year)		Single Use Only
	Product Lot Information		Expiration date		CE Symbol
	Reference No		Unique Device ID		Medical Device
	Do not expose to sunlight		Keep Dry		Store between 0°C and 40°C
	Max. 70% RH		Metal free		Lead free
	Suitable for steam sterilization		Suitable for Ethylene Oxide sterilization		Suitable for formaldehyde sterilization

### Packaging and Product Information:

Product Code	Product Description	Product Size	Pcs In Box*
SRZ0505	Flat Sterilization Pouch with 2 indicators	5cm x 5cm	5000/2000
SRZ0575	Flat Sterilization Pouch with 2 indicators	5cm x 7,5cm	5000/2000
SRZ0510	Flat Sterilization Pouch with 2 indicators	5cm x 10cm	5000/2000
SRZ0515	Flat Sterilization Pouch with 2 indicators	5cm x 15cm	5000/2000
SRZ0520	Flat Sterilization Pouch with 2 indicators	5cm x 20cm	5000/2000
SRZ0525	Flat Sterilization Pouch with 2 indicators	5cm x 25cm	5000/2000
SRZ0530	Flat Sterilization Pouch with 2 indicators	5cm x 30cm	5000/2000
SRZ0535	Flat Sterilization Pouch with 2 indicators	5cm x 35cm	5000/2000
SRZ0540	Flat Sterilization Pouch with 2 indicators	5cm x 40cm	5000/2000
SRZ0545	Flat Sterilization Pouch with 2 indicators	5cm x 45cm	5000/2000
SRZ0550	Flat Sterilization Pouch with 2 indicators	5cm x 50cm	5000/2000
SRZ7575	Flat Sterilization Pouch with 2 indicators	7,5cm x 7,5cm	5000/2000
SRZ7510	Flat Sterilization Pouch with 2 indicators	7,5cm x 10cm	5000/2000
SRZ7515	Flat Sterilization Pouch with 2 indicators	7,5cm x 15cm	5000/2000
SRZ7520	Flat Sterilization Pouch with 2 indicators	7,5cm x 20cm	5000/2000
SRZ7525	Flat Sterilization Pouch with 2 indicators	7,5cm x 25cm	5000/2000



Medster Tıbbi Cihaz ve Sağlık Hizmetleri LTD. ŞTİ.

Başkent OSB Malıköy 19. Cad No.51 Sincan Ankara TURKEY

Tel: +90 312 387 05 50 Fax: +90 312 387 05 51 www.medster.com.tr

## INSTRUCTION FOR USE

SRZ7530	Flat Sterilization Pouch with 2 indicators	7,5cm x 30cm	5000/2000
SRZ7535	Flat Sterilization Pouch with 2 indicators	7,5cm x 35cm	5000/2000
SRZ7540	Flat Sterilization Pouch with 2 indicators	7,5cm x 40cm	5000/2000
SRZ7545	Flat Sterilization Pouch with 2 indicators	7,5cm x 45cm	5000/2000
SRZ7550	Flat Sterilization Pouch with 2 indicators	7,5cm x 50cm	5000/2000
SRZ1010	Flat Sterilization Pouch with 2 indicators	10cm x 10cm	5000/2000
SRZ1015	Flat Sterilization Pouch with 2 indicators	10cm x 15cm	5000/2000
SRZ1020	Flat Sterilization Pouch with 2 indicators	10cm x 20cm	5000/2000
SRZ1025	Flat Sterilization Pouch with 2 indicators	10cm x 25cm	5000/2000
SRZ1030	Flat Sterilization Pouch with 2 indicators	10cm x 30cm	5000/2000
SRZ1035	Flat Sterilization Pouch with 2 indicators	10cm x 35cm	5000/2000
SRZ1040	Flat Sterilization Pouch with 2 indicators	10cm x 40cm	5000/2000
SRZ1045	Flat Sterilization Pouch with 2 indicators	10cm x 45cm	5000/2000
SRZ1050	Flat Sterilization Pouch with 2 indicators	10cm x 50cm	5000/2000
SRZ1515	Flat Sterilization Pouch with 2 indicators	15cm x 15cm	5000/2000
SRZ1520	Flat Sterilization Pouch with 2 indicators	15cm x 20cm	5000/2000
SRZ1525	Flat Sterilization Pouch with 2 indicators	15cm x 25cm	5000/2000
SRZ1530	Flat Sterilization Pouch with 2 indicators	15cm x 30cm	5000/2000
SRZ1535	Flat Sterilization Pouch with 2 indicators	15cm x 35cm	5000/2000
SRZ1540	Flat Sterilization Pouch with 2 indicators	15cm x 40cm	5000/2000
SRZ1545	Flat Sterilization Pouch with 2 indicators	15cm x 45cm	5000/2000
SRZ1550	Flat Sterilization Pouch with 2 indicators	15cm x 50cm	5000/2000
SRZ2020	Flat Sterilization Pouch with 2 indicators	20cm x 20cm	5000/2000
SRZ2025	Flat Sterilization Pouch with 2 indicators	20cm x 25cm	5000/2000
SRZ2030	Flat Sterilization Pouch with 2 indicators	20cm x 30cm	5000/2000
SRZ2035	Flat Sterilization Pouch with 2 indicators	20cm x 35cm	2000/1000
SRZ2040	Flat Sterilization Pouch with 2 indicators	20cm x 40cm	2000/1000
SRZ2045	Flat Sterilization Pouch with 2 indicators	20cm x 45cm	2000/1000
SRZ2050	Flat Sterilization Pouch with 2 indicators	20cm x 50cm	2000/1000
SRZ2525	Flat Sterilization Pouch with 2 indicators	25cm x 25cm	2000/1000
SRZ2530	Flat Sterilization Pouch with 2 indicators	25cm x 30cm	2000/1000
SRZ2535	Flat Sterilization Pouch with 2 indicators	25cm x 35cm	2000/1000
SRZ2540	Flat Sterilization Pouch with 2 indicators	25cm x 40cm	2000/1000
SRZ2545	Flat Sterilization Pouch with 2 indicators	25cm x 45cm	2000/1000
SRZ2550	Flat Sterilization Pouch with 2 indicators	25cm x 50cm	2000/1000
SRZ3030	Flat Sterilization Pouch with 2 indicators	30cm x 30cm	2000/1000
SRZ3035	Flat Sterilization Pouch with 2 indicators	30cm x 35cm	2000/1000



## INSTRUCTION FOR USE

SRZ3040	Flat Sterilization Pouch with 2 indicators	30cm x 40cm	2000/1000
SRZ3045	Flat Sterilization Pouch with 2 indicators	30cm x 45cm	2000/1000
SRZ3050	Flat Sterilization Pouch with 2 indicators	30cm x 50cm	2000/1000
SRZ3535	Flat Sterilization Pouch with 2 indicators	35cm x 35cm	2000/1000
SRZ3540	Flat Sterilization Pouch with 2 indicators	35cm x 40cm	2000/1000
SRZ3545	Flat Sterilization Pouch with 2 indicators	35cm x 45cm	2000/1000
SRZ3550	Flat Sterilization Pouch with 2 indicators	35cm x 50cm	2000/1000
SRZ4040	Flat Sterilization Pouch with 2 indicators	40cm x 40cm	2000/1000
SRZ4045	Flat Sterilization Pouch with 2 indicators	40cm x 45cm	2000/1000
SRZ4050	Flat Sterilization Pouch with 2 indicators	40cm x 50cm	2000/1000
SRZ4545	Flat Sterilization Pouch with 2 indicators	45cm x 45cm	2000/1000
SRZ4550	Flat Sterilization Pouch with 2 indicators	45cm x 50cm	2000/1000
SRZ5050	Flat Sterilization Pouch with 2 indicators	50cm x 50cm	2000/1000
SRZ05175	Flat Sterilization Pouch with 2 indicators	5cmx17,5cm	5000/2000
SRZ05225	Flat Sterilization Pouch with 2 indicators	5cmx22,5cm	5000/2000
SRZ75175	Flat Sterilization Pouch with 2 indicators	7,5cmx17,5cm	5000/2000
SRZ75225	Flat Sterilization Pouch with 2 indicators	7,5cmx22,5cm	5000/2000
SRZ3055	Flat Sterilization Pouch with 2 indicators	30cmx55cm	2000
SRZ3060	Flat Sterilization Pouch with 2 indicators	30cmx60cm	2000
SRZ10175	Flat Sterilization Pouch with 2 indicators	10cmx17,5cm	5000/2000
SRZ15175	Flat Sterilization Pouch with 2 indicators	15cmx17,5cm	5000/2000
SRZ3555	Flat Sterilization Pouch with 2 indicators	35cmx55cm	2000/1000
SRZ3560	Flat Sterilization Pouch with 2 indicators	35cmx60cm	2000/1000
SRZ4055	Flat Sterilization Pouch with 2 indicators	40cmx55cm	2000/1000
SRZ4060	Flat Sterilization Pouch with 2 indicators	40cmx60cm	2000/1000
SRZ10225	Flat Sterilization Pouch with 2 indicators	10cmx22,5cm	5000/2000
SRZ15225	Flat Sterilization Pouch with 2 indicators	15cmx22,5cm	5000/2000
SRZ20225	Flat Sterilization Pouch with 2 indicators	20cmx22,5cm	5000/2000
SRZ4555	Flat Sterilization Pouch with 2 indicators	45cmx55cm	2000/1000
SRZ4560	Flat Sterilization Pouch with 2 indicators	45cmx60cm	2000/1000
SRZ1512	Flat Sterilization Pouch with 2 indicators	15cmx12cm	5000/2000
SRZ0505-1	Flat Sterilization Pouch with 1 indicator	5cm x 5cm	5000/2000
SRZ0575-1	Flat Sterilization Pouch with 1 indicator	5cm x 7,5cm	5000/2000
SRZ0510-1	Flat Sterilization Pouch with 1 indicator	5cm x 10cm	5000/2000
SRZ0515-1	Flat Sterilization Pouch with 1 indicator	5cm x 15cm	5000/2000
SRZ0520-1	Flat Sterilization Pouch with 1 indicator	5cm x 20cm	5000/2000
SRZ0525-1	Flat Sterilization Pouch with 1 indicator	5cm x 25cm	5000/2000



## INSTRUCTION FOR USE

SRZ0530-1	Flat Sterilization Pouch with 1 indicator	5cm x 30cm	5000/2000
SRZ0535-1	Flat Sterilization Pouch with 1 indicator	5cm x 35cm	5000/2000
SRZ0540-1	Flat Sterilization Pouch with 1 indicator	5cm x 40cm	5000/2000
SRZ0545-1	Flat Sterilization Pouch with 1 indicator	5cm x 45cm	5000/2000
SRZ0550-1	Flat Sterilization Pouch with 1 indicator	5cm x 50cm	5000/2000
SRZ7575-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 7,5cm	5000/2000
SRZ7510-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 10cm	5000/2000
SRZ7515-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 15cm	5000/2000
SRZ7520-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 20cm	5000/2000
SRZ7525-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 25cm	5000/2000
SRZ7530-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 30cm	5000/2000
SRZ7535-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 35cm	5000/2000
SRZ7540-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 40cm	5000/2000
SRZ7545-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 45cm	5000/2000
SRZ7550-1	Flat Sterilization Pouch with 1 indicator	7,5cm x 50cm	5000/2000
SRZ1010-1	Flat Sterilization Pouch with 1 indicator	10cm x 10cm	5000/2000
SRZ1015-1	Flat Sterilization Pouch with 1 indicator	10cm x 15cm	5000/2000
SRZ1020-1	Flat Sterilization Pouch with 1 indicator	10cm x 20cm	5000/2000
SRZ1025-1	Flat Sterilization Pouch with 1 indicator	10cm x 25cm	5000/2000
SRZ1030-1	Flat Sterilization Pouch with 1 indicator	10cm x 30cm	5000/2000
SRZ1035-1	Flat Sterilization Pouch with 1 indicator	10cm x 35cm	5000/2000
SRZ1040-1	Flat Sterilization Pouch with 1 indicator	10cm x 40cm	5000/2000
SRZ1045-1	Flat Sterilization Pouch with 1 indicator	10cm x 45cm	5000/2000
SRZ1050-1	Flat Sterilization Pouch with 1 indicator	10cm x 50cm	5000/2000
SRZ1515-1	Flat Sterilization Pouch with 1 indicator	15cm x 15cm	5000/2000
SRZ1520-1	Flat Sterilization Pouch with 1 indicator	15cm x 20cm	5000/2000
SRZ1525-1	Flat Sterilization Pouch with 1 indicator	15cm x 25cm	5000/2000
SRZ1530-1	Flat Sterilization Pouch with 1 indicator	15cm x 30cm	5000/2000
SRZ1535-1	Flat Sterilization Pouch with 1 indicator	15cm x 35cm	5000/2000
SRZ1540-1	Flat Sterilization Pouch with 1 indicator	15cm x 40cm	5000/2000
SRZ1545-1	Flat Sterilization Pouch with 1 indicator	15cm x 45cm	5000/2000
SRZ1550-1	Flat Sterilization Pouch with 1 indicator	15cm x 50cm	5000/2000
SRZ2020-1	Flat Sterilization Pouch with 1 indicator	20cm x 20cm	5000/2000
SRZ2025-1	Flat Sterilization Pouch with 1 indicator	20cm x 25cm	5000/2000
SRZ2030-1	Flat Sterilization Pouch with 1 indicator	20cm x 30cm	5000/2000
SRZ2035-1	Flat Sterilization Pouch with 1 indicator	20cm x 35cm	2000/1000
SRZ2040-1	Flat Sterilization Pouch with 1 indicator	20cm x 40cm	2000/1000



## INSTRUCTION FOR USE

SRZ2045-1	Flat Sterilization Pouch with 1 indicator	20cm x 45cm	2000/1000
SRZ2050-1	Flat Sterilization Pouch with 1 indicator	20cm x 50cm	2000/1000
SRZ2525-1	Flat Sterilization Pouch with 1 indicator	25cm x 25cm	2000/1000
SRZ2530-1	Flat Sterilization Pouch with 1 indicator	25cm x 30cm	2000/1000
SRZ2535-1	Flat Sterilization Pouch with 1 indicator	25cm x 35cm	2000/1000
SRZ2540-1	Flat Sterilization Pouch with 1 indicator	25cm x 40cm	2000/1000
SRZ2545-1	Flat Sterilization Pouch with 1 indicator	25cm x 45cm	2000/1000
SRZ2550-1	Flat Sterilization Pouch with 1 indicator	25cm x 50cm	2000/1000
SRZ3030-1	Flat Sterilization Pouch with 1 indicator	30cm x 30cm	2000/1000
SRZ3035-1	Flat Sterilization Pouch with 1 indicator	30cm x 35cm	2000/1000
SRZ3040-1	Flat Sterilization Pouch with 1 indicator	30cm x 40cm	2000/1000
SRZ3045-1	Flat Sterilization Pouch with 1 indicator	30cm x 45cm	2000/1000
SRZ3050-1	Flat Sterilization Pouch with 1 indicator	30cm x 50cm	2000/1000
SRZ3535-1	Flat Sterilization Pouch with 1 indicator	35cm x 35cm	2000/1000
SRZ3540-1	Flat Sterilization Pouch with 1 indicator	35cm x 40cm	2000/1000
SRZ3545-1	Flat Sterilization Pouch with 1 indicator	35cm x 45cm	2000/1000
SRZ3550-1	Flat Sterilization Pouch with 1 indicator	35cm x 50cm	2000/1000
SRZ4040-1	Flat Sterilization Pouch with 1 indicator	40cm x 40cm	2000/1000
SRZ4045-1	Flat Sterilization Pouch with 1 indicator	40cm x 45cm	2000/1000
SRZ4050-1	Flat Sterilization Pouch with 1 indicator	40cm x 50cm	2000/1000
SRZ4545-1	Flat Sterilization Pouch with 1 indicator	45cm x 45cm	2000/1000
SRZ4550-1	Flat Sterilization Pouch with 1 indicator	45cm x 50cm	2000/1000
SRZ5050-1	Flat Sterilization Pouch with 1 indicator	50cm x 50cm	2000/1000
SRZ05175-1	Flat Sterilization Pouch with 1 indicator	5cmx17,5cm	5000/2000
SRZ05225-1	Flat Sterilization Pouch with 1 indicator	5cmx22,5cm	5000/2000
SRZ75175-1	Flat Sterilization Pouch with 1 indicator	7,5cmx17,5cm	5000/2000
SRZ75225-1	Flat Sterilization Pouch with 1 indicator	7,5cmx22,5cm	5000/2000
SRZ3055-1	Flat Sterilization Pouch with 1 indicator	30cmx55cm	2000/1000
SRZ3060-1	Flat Sterilization Pouch with 1 indicator	30cmx60cm	2000/1000
SRZ10175-1	Flat Sterilization Pouch with 1 indicator	10cmx17,5cm	5000/2000
SRZ15175-1	Flat Sterilization Pouch with 1 indicator	15cmx17,5cm	5000/2000
SRZ3555-1	Flat Sterilization Pouch with 1 indicator	35cmx55cm	2000/1000
SRZ3560-1	Flat Sterilization Pouch with 1 indicator	35cmx60cm	2000/1000
SRZ4055-1	Flat Sterilization Pouch with 1 indicator	40cmx55cm	2000/1000
SRZ4060-1	Flat Sterilization Pouch with 1 indicator	40cmx60cm	2000/1000
SRZ10225-1	Flat Sterilization Pouch with 1 indicator	10cmx22,5cm	5000/2000
SRZ15225-1	Flat Sterilization Pouch with 1 indicator	15cmx22,5cm	5000/2000



## INSTRUCTION FOR USE

SRZ20225-1	Flat Sterilization Pouch with 1 indicator	20cmx22,5cm	5000/2000
SRZ4555-1	Flat Sterilization Pouch with 1 indicator	45cmx55cm	2000/1000
SRZ4560-1	Flat Sterilization Pouch with 1 indicator	45cmx60cm	2000/1000
SRZ1512-1	Flat Sterilization Pouch with 1 indicator	15cmx12cm	5000/2000
SRZ0505-3	Flat Sterilization Pouch with 3 indicators	5cm x 5cm	5000/2000
SRZ0575-3	Flat Sterilization Pouch with 3 indicators	5cm x 7,5cm	5000/2000
SRZ0510-3	Flat Sterilization Pouch with 3 indicators	5cm x 10cm	5000/2000
SRZ0515-3	Flat Sterilization Pouch with 3 indicators	5cm x 15cm	5000/2000
SRZ0520-3	Flat Sterilization Pouch with 3 indicators	5cm x 20cm	5000/2000
SRZ0525-3	Flat Sterilization Pouch with 3 indicators	5cm x 25cm	5000/2000
SRZ0530-3	Flat Sterilization Pouch with 3 indicators	5cm x 30cm	5000/2000
SRZ0535-3	Flat Sterilization Pouch with 3 indicators	5cm x 35cm	5000/2000
SRZ0540-3	Flat Sterilization Pouch with 3 indicators	5cm x 40cm	5000/2000
SRZ0545-3	Flat Sterilization Pouch with 3 indicators	5cm x 45cm	5000/2000
SRZ0550-3	Flat Sterilization Pouch with 3 indicators	5cm x 50cm	5000/2000
SRZ7575-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 7,5cm	5000/2000
SRZ7510-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 10cm	5000/2000
SRZ7515-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 15cm	5000/2000
SRZ7520-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 20cm	5000/2000
SRZ7525-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 25cm	5000/2000
SRZ7530-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 30cm	5000/2000
SRZ7535-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 35cm	5000/2000
SRZ7540-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 40cm	5000/2000
SRZ7545-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 45cm	5000/2000
SRZ7550-3	Flat Sterilization Pouch with 3 indicators	7,5cm x 50cm	5000/2000
SRZ1010-3	Flat Sterilization Pouch with 3 indicators	10cm x 10cm	5000/2000
SRZ1015-3	Flat Sterilization Pouch with 3 indicators	10cm x 15cm	5000/2000
SRZ1020-3	Flat Sterilization Pouch with 3 indicators	10cm x 20cm	5000/2000
SRZ1025-3	Flat Sterilization Pouch with 3 indicators	10cm x 25cm	5000/2000
SRZ1030-3	Flat Sterilization Pouch with 3 indicators	10cm x 30cm	5000/2000
SRZ1035-3	Flat Sterilization Pouch with 3 indicators	10cm x 35cm	5000/2000
SRZ1040-3	Flat Sterilization Pouch with 3 indicators	10cm x 40cm	5000/2000
SRZ1045-3	Flat Sterilization Pouch with 3 indicators	10cm x 45cm	5000/2000
SRZ1050-3	Flat Sterilization Pouch with 3 indicators	10cm x 50cm	5000/2000
SRZ1515-3	Flat Sterilization Pouch with 3 indicators	15cm x 15cm	5000/2000
SRZ1520-3	Flat Sterilization Pouch with 3 indicators	15cm x 20cm	5000/2000
SRZ1525-3	Flat Sterilization Pouch with 3 indicators	15cm x 25cm	5000/2000



## INSTRUCTION FOR USE

SRZ1530-3	Flat Sterilization Pouch with 3 indicators	15cm x 30cm	5000/2000
SRZ1535-3	Flat Sterilization Pouch with 3 indicators	15cm x 35cm	5000/2000
SRZ1540-3	Flat Sterilization Pouch with 3 indicators	15cm x 40cm	5000/2000
SRZ1545-3	Flat Sterilization Pouch with 3 indicators	15cm x 45cm	5000/2000
SRZ1550-3	Flat Sterilization Pouch with 3 indicators	15cm x 50cm	5000/2000
SRZ2020-3	Flat Sterilization Pouch with 3 indicators	20cm x 20cm	5000/2000
SRZ2025-3	Flat Sterilization Pouch with 3 indicators	20cm x 25cm	5000/2000
SRZ2030-3	Flat Sterilization Pouch with 3 indicators	20cm x 30cm	5000/2000
SRZ2035-3	Flat Sterilization Pouch with 3 indicators	20cm x 35cm	2000/1000
SRZ2040-3	Flat Sterilization Pouch with 3 indicators	20cm x 40cm	2000/1000
SRZ2045-3	Flat Sterilization Pouch with 3 indicators	20cm x 45cm	2000/1000
SRZ2050-3	Flat Sterilization Pouch with 3 indicators	20cm x 50cm	2000/1000
SRZ2525-3	Flat Sterilization Pouch with 3 indicators	25cm x 25cm	2000/1000
SRZ2530-3	Flat Sterilization Pouch with 3 indicators	25cm x 30cm	2000/1000
SRZ2535-3	Flat Sterilization Pouch with 3 indicators	25cm x 35cm	2000/1000
SRZ2540-3	Flat Sterilization Pouch with 3 indicators	25cm x 40cm	2000/1000
SRZ2545-3	Flat Sterilization Pouch with 3 indicators	25cm x 45cm	2000/1000
SRZ2550-3	Flat Sterilization Pouch with 3 indicators	25cm x 50cm	2000/1000
SRZ3030-3	Flat Sterilization Pouch with 3 indicators	30cm x 30cm	2000/1000
SRZ3035-3	Flat Sterilization Pouch with 3 indicators	30cm x 35cm	2000/1000
SRZ3040-3	Flat Sterilization Pouch with 3 indicators	30cm x 40cm	2000/1000
SRZ3045-3	Flat Sterilization Pouch with 3 indicators	30cm x 45cm	2000/1000
SRZ3050-3	Flat Sterilization Pouch with 3 indicators	30cm x 50cm	2000/1000
SRZ3535-3	Flat Sterilization Pouch with 3 indicators	35cm x 35cm	2000/1000
SRZ3540-3	Flat Sterilization Pouch with 3 indicators	35cm x 40cm	2000/1000
SRZ3545-3	Flat Sterilization Pouch with 3 indicators	35cm x 45cm	2000/1000
SRZ3550-3	Flat Sterilization Pouch with 3 indicators	35cm x 50cm	2000/1000
SRZ4040-3	Flat Sterilization Pouch with 3 indicators	40cm x 40cm	2000/1000
SRZ4045-3	Flat Sterilization Pouch with 3 indicators	40cm x 45cm	2000/1000
SRZ4050-3	Flat Sterilization Pouch with 3 indicators	40cm x 50cm	2000/1000
SRZ4545-3	Flat Sterilization Pouch with 3 indicators	45cm x 45cm	2000/1000
SRZ4550-3	Flat Sterilization Pouch with 3 indicators	45cm x 50cm	2000/1000
SRZ5050-3	Flat Sterilization Pouch with 3 indicators	50cm x 50cm	2000/1000
SRZ05175-3	Flat Sterilization Pouch with 3 indicators	5cm x 17,5cm	5000/2000
SRZ05225-3	Flat Sterilization Pouch with 3 indicators	5cm x 22,5cm	5000/2000
SRZ75175-3	Flat Sterilization Pouch with 3 indicators	7,5cmx17,5cm	5000/2000
SRZ75225-3	Flat Sterilization Pouch with 3 indicators	7,5cmx22,5cm	5000/2000





## INSTRUCTION FOR USE

SRZ3055-3	Flat Sterilization Pouch with 3 indicators	30cmx55cm	2000/1000
SRZ3060-3	Flat Sterilization Pouch with 3 indicators	30cmx60cm	2000/1000
SRZ10175-3	Flat Sterilization Pouch with 3 indicators	10cmx17,5cm	5000/2000
SRZ15175-3	Flat Sterilization Pouch with 3 indicators	15cmx17,5cm	5000/2000
SRZ3555-3	Flat Sterilization Pouch with 3 indicators	35cmx55cm	2000/1000
SRZ3560-3	Flat Sterilization Pouch with 3 indicators	35cmx60cm	2000/1000
SRZ4055-3	Flat Sterilization Pouch with 3 indicators	40cmx55cm	2000/1000
SRZ4060-3	Flat Sterilization Pouch with 3 indicators	40cmx60cm	2000/1000
SRZ10225-3	Flat Sterilization Pouch with 3 indicators	10cmx22,5cm	5000/2000
SRZ15225-3	Flat Sterilization Pouch with 3 indicators	15cmx22,5cm	5000/2000
SRZ20225-3	Flat Sterilization Pouch with 3 indicators	20cmx22,5cm	5000/2000
SRZ4555-3	Flat Sterilization Pouch with 3 indicators	45cmx55cm	2000/1000
SRZ4560-3	Flat Sterilization Pouch with 3 indicators	45cmx60cm	2000/1000
SRZ1512-3	Flat Sterilization Pouch with 3 indicators	15cmx12cm	5000/2000

\*Packing with 2 options.



Medster Tıbbi Cihaz ve Saęlık Hizmetleri LTD. ŐTİ.

BaŐkent OSB Malıköy 19. Cad No.51 Sincan Ankara TURKEY

Tel: +90 312 387 05 50 Fax: +90 312 387 05 51 www.medster.com.tr