


Advantages

- ✓ test soil contains sheep blood according to EN ISO 15883-5
- ✓ simulates 4 instrument categories in 1 Test
- ✓ the unique round design simulates different cleaning angles
- ✓ easy handling and visual monitoring possible
- ✓ complete documentation

Description

The Josafe CCI easy simulates 4 different categories of surgical instruments in 4 different fields.

With only 1 test the Josafe CCI simulates the cleaning effectivity of MIC and eye instruments, forceps, scissors and general medical equipment.

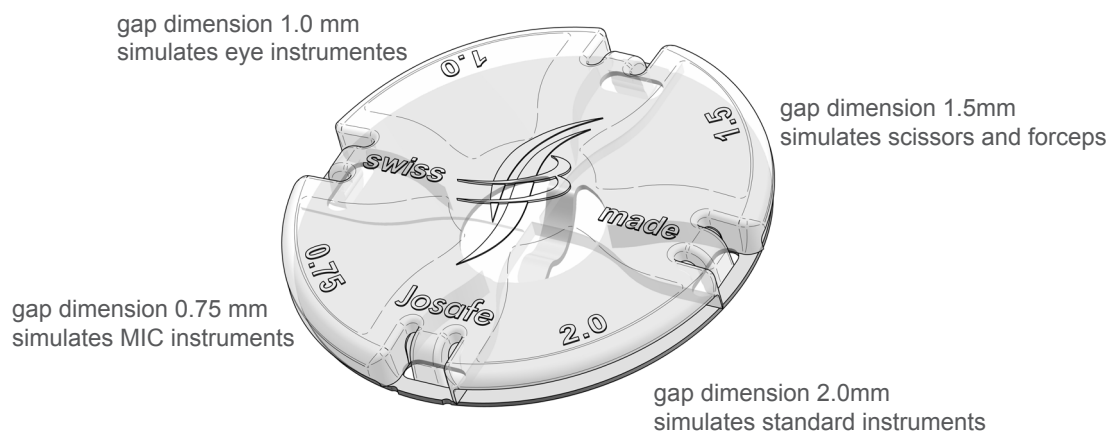

SPECIFICATIONS

Application:	cleaning indicator
Norm:	ISO 15883-5
Storage:	5°-30°C
Shelf live:	2 years
Not toxic:	yes
Article code:	J0024 single pack J0010 set with 10 single packs
Packing unit:	box with 24 pcs

Routine check washer disinfector according to the norm and guidelines

	VALIDATED WD	NOT VALIDATED WD	NOT VALIDATABLE WD
CLEANING INDICATOR CCI	every 4 weeks	weekly	daily

Place an indicator on every load level. Choose the position randomly. Make sure that the position is allways different.




positive result






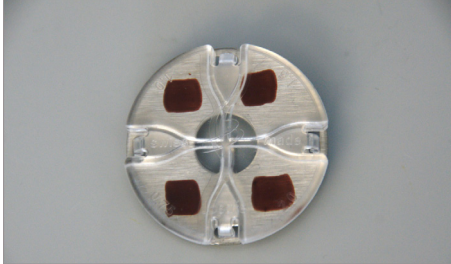
negative result



negative result

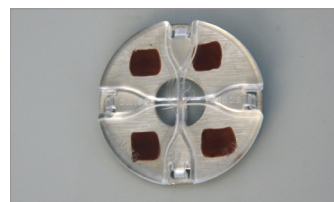

negative result

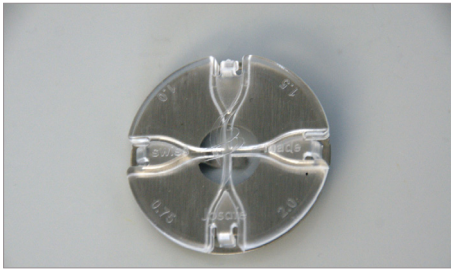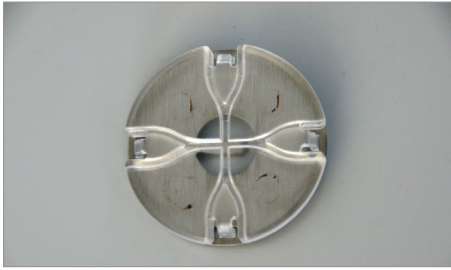RESULT	POSSIBLE CAUSES	ACTION
	Soil completely removed positiv result	release batch!
	Soil partially removed negative result - spray shadows - incorrect load - spray pressure is too low	Do not release batch! - check load, program, spray arms, etc. - repeat test
	Soil largely preserved negative result - spray shadows - incorrect load - spray pressure is too low	Do not release batch! - check load, program, spray arms, etc. - repeat test - detergent flow check
	Soil completely preserved negative result => no cleaning effect - spray shadows - incorrect load - spray pressure is too low - temperature not reached	Do not release batch! - Lock machine - check load, program, spray arms, etc. - repeat test - detergent flow check - call engineer

Routine check washer disinfector according to the norm and guidelines

	VALIDATED WD	NOT VALIDATED WD	NOT VALIDATABLE WD
CLEANING INDICATOR CCI	every 4 weeks	weekly	daily

Place an indicator on every load level. Choose the position randomly. Make sure that the position is always different.